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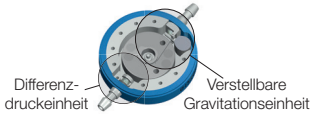
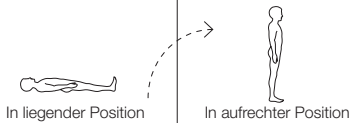
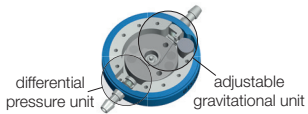
Patient	Selection of pressure levels		Opening pressure in total		
		differential pressure unit adjustable gravitational unit	horizontal position	upright position	
Newborn 			5*	20	5*
Children from 3 years 	30				
Adults < 1,60 m > 1,80 m 	25	30			
	20	25			
30	35				
Adults from 65 years 	20	15	25	25	
< 1,60 m				20	
> 1,80 m				30	

Standard pressure levels for *M.blue*® and *M.blue plus*®

The *M.blue* consists of a differential pressure unit and an adjustable gravitational unit.

The opening pressure of *M.blue* in the horizontal position is **only** determined by the differential pressure unit.

In the lying position the gravitational unit is not active.

The opening pressure in the vertical position is **the sum** of both: differential pressure unit and adjustable gravitational unit.

The Miethke App demonstrates the functionality of *M.blue* and the effect of patient posture on valve opening pressure.

Now available in the App and Google Play Store.

All indicated pressure levels are in cmH₂O. This recommendation is non-binding. The physician decides in each case individually. The stated values consider the current scientific knowledge up to June 2018.

* As special configurations in addition to the standard configuration the *M.blue* is also available with differential pressure units with pressure levels 0, 10, 15 cmH₂O. In case of *M.blue plus* the opening pressure of the differential pressure unit (*proGAV 2.0*) is adjustable from 0-20 cmH₂O.

Manufacturer:

CHRISTOPH MIETHKE GMBH & CO. KG

Ulanenweg 2 | 14469 Potsdam | Germany

Phone +49 331 62 083-0 | Fax +49 331 62 083-40

www.miethke.com

Distributor:

B | BRAUN
SHARING EXPERTISE

AESCLAP® – a B. Braun brand

Aesculap AG | Am Aesculap-Platz | 78532 Tuttlingen | Germany

Phone +49 7461 95-0 | Fax +49 7461 95-26 00 | www.aesculap.com

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Aesculap AG | Am Aesculap-Platz | 78532 Tuttlingen | Deutschland